

Surgical Expert Systems:

# Acute Kidney Injury

Acute Kidney Injury (AKI) is a highly pervasive condition, particularly amongst hospitalized patients<sup>1</sup>. The complication involves a sudden and often reversible reduction in kidney function, as measured by increased creatinine or decreased urine volume<sup>1</sup>. Even a minor increase in serum creatinine from baseline is independently associated with increased mortality and contributes to long-term morbidity<sup>2</sup>. Risk factors for AKI include male sex, advanced age, increased BMI, and hypertension<sup>3</sup>.

*Each year, approximately 1.7 million people around the world die from AKI<sup>4</sup>.*

**AKI risk scores provided by Stream Care™ are selected based on a thorough and extensive review of existing literature, incorporating:**

- ✓ 42 Peer Reviewed Papers
- ✓ 2 Systematic Reviews
- ✓ 1 Textbook



## Impact

AKI affects approximately 10–20% of hospitalized patients, but incidence is as high as 45–50% among critically ill patients<sup>5</sup>. In colorectal surgical populations, postoperative AKI (PO-AKI) incidence ranges from 3.8–19.5%<sup>6</sup>. Similarly, 3%–17.5% of bariatric patients and 2.4–14.7% of esophageal experience the complication postoperatively<sup>7–10</sup>.

AKI is associated with significant morbidity, mortality, extra costs incurred in the hospitalization process, longer stay in the hospital, and long-term consequences, including chronic kidney disease (CKD) and end-stage kidney disease (ESKD)<sup>5</sup>. Overall mortality rate at 30 days following the complication is as high as 24%<sup>11</sup>. Furthermore, PO-AKI is associated with electrolyte imbalance, fluid accumulation, and metabolic dysfunctions, leading to a cascade of cardiovascular, respiratory, neurological, infectious, and coagulation disorders<sup>12</sup>. In the United States, AKI is associated with high hospitalization costs that range from \$5.4 to \$24.0 billion<sup>13</sup>.

# Driving *timely intervention* with early automated prediction of AKI.

## Static Risk Scores

### SPARK

The Simple Postoperative AKI Risk (SPARK) score **predicts postoperative AKI** in noncardiac surgery patients and is **calculated in preoperative periods**<sup>14</sup>.

#### Source

SPARK was developed by [Park et al.](#) and validated by [Li et al.](#) and [Nishimoto et al.](#)

#### Patient Population

SPARK was developed using patients who underwent a noncardiac operation<sup>14</sup>.

#### Data Set

Seoul National University Hospital<sup>14</sup>

#### Sample Size

51,041<sup>14</sup>

#### Inputs

- Age
- eGFR
- Urine Albumin
- Sex
- Hypoalbuminemia
- Emergency Operation
- Hypoatremia
- Surgical Duration
- Diabetes
- RAAS Blockade Use
- Anemia

## Performance Metrics

Risk Score	Cited By	Reference	Validation Type	AUC	Specificity	Sensitivity	NPV	PPV
SPARK	99	<a href="#">Park et al.</a>	Internal	0.81	0.75	0.82	–	–
		<a href="#">Li et al.</a>	External	0.703	0.5749	0.6835	0.9536	0.1244
		<a href="#">Nishimoto et al.</a>	External	0.67	–	–	–	–

## References

1. Goyal A, Daneshpajouhnejad P, Hashmi MF, Bashir K. Acute Kidney Injury. In: StatPearls. StatPearls Publishing; 2025. Accessed April 1, 2025. <http://www.ncbi.nlm.nih.gov/books/NBK441896/>
2. Wijdicks EFM. Acute Kidney Injury. In: Wijdicks EFM, Wijdicks EFM, eds. The Practice of Emergency and Critical Care Neurology. Oxford University Press; 2025:0. doi:10.1093/med/9780197544976.003.0056
3. Huang L, Xiao A, Li Y. Risk factors for postoperative acute kidney injury in colorectal cancer: a systematic review and meta-analysis. Int J Colorectal Dis. 2025;40(1):70. doi:10.1007/s00384-025-04860-7
4. Mehta RL, Cerdá J, Burdmann EA, et al. International Society of Nephrology's Oby25 initiative for acute kidney injury (zero preventable deaths by 2025): a human rights case for nephrology. The Lancet. 2015;385(9987):2616-2643. doi:10.1016/S0140-6736(15)60126-X
5. Thongprayoon C, Hansrivijit P, Kovvuru K, et al. Diagnostics, Risk Factors, Treatment and Outcomes of Acute Kidney Injury in a New Paradigm. J Clin Med. 2020;9(4):1104. doi:10.3390/jcm9041104
6. Acute kidney injury is a common and significant complication following ileostomy formation. doi:10.1111/codi.15917
7. Parab SY, Majety SC, Ranganathan P, Jiwnani S, Pramesh C, Shetmahajan M. Incidence of acute kidney injury and its associated risk factors in patients undergoing elective oesophagectomy surgeries at a tertiary care cancer institute – A pilot prospective observational study. Indian J Anaesth. 2024;68(6):572-578. doi:10.4103/ija.ija\_98\_24
8. A289 Assessing Acute Kidney Injury Risks Following Bariatric Surgery: A 10-Year Update. Surg Obes Relat Dis. 2024;20(6, Supplement):S91. doi:10.1016/j.soard.2024.04.289
9. Morgan DJR, Ho KM. Acute kidney injury in bariatric surgery patients requiring intensive care admission: a state-wide, multicenter, cohort study. Surg Obes Relat Dis Off J Am Soc Bariat Surg. 2015;11(6):1300-1306. doi:10.1016/j.soard.2015.01.005
10. Wang W, Wang T, Feng X, Sun L. Incidence and risk factors of acute kidney injury after esophageal cancer surgery: A nested case-control study. Int J Surg. 2017;39:11-15. doi:10.1016/j.ijsu.2017.01.043
11. Defining the Cause of Death in Hospitalised Patients with Acute Kidney Injury | PLOS One. Accessed April 1, 2025. <https://journals.plos.org/plosone/article?id=10.1371/journal.pone.0048580>
12. Brusasco C, Cuccioli G, Barberis A, Intorini C, Campodonico F, Corradi F. Improving Patient Outcomes in Abdominal Surgery. J Clin Med. 2024;13(7):1993. doi:10.3390/jcm13071993
13. Silver SA, Chertow GM. The Economic Consequences of Acute Kidney Injury. Nephron. 2017;137(4):297-301. doi:10.1159/000475607
14. Park S, Cho H, Park S, et al. Simple Postoperative AKI Risk (SPARK) Classification before Noncardiac Surgery: A Prediction Index Development Study with External Validation. J Am Soc Nephrol JASN. 2019;30(1):170-181. doi:10.1681/ASN.2018070757



**FluidAI Medical**  
809 Wellington St N Unit 2,  
Kitchener, ON N2H 5L6

info@fluidai.md  
www.fluidai.md  
+1 (877) 660-6378